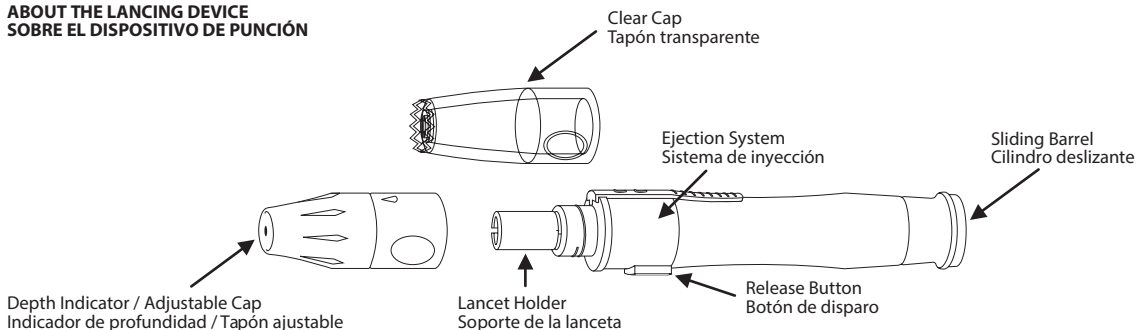
INSTRUCTIONS FOR MYSTAR SYLKFEEL® LANCING DEVICE

The **MyStar SylkFeel®** lancing device is a medical tool for collecting capillary blood samples for glucose monitoring or other tests which require one or two small drops of blood. The **MyStar SylkFeel®** lancing device is used to get samples of blood for testing using a lancet. The lancing device has a spring driven mechanism to enable an automated lancing process which is released using a spring-loaded trigger button. The lancing device can be set to puncture the skin at 5 different levels. In addition, the **MyStar SylkFeel®** also may come with a transparent cap for Alternative Site Testing (AST), check your blood glucose meter instructions if AST is allowed.

PRECAUTIONS

- The MyStar SylkFeel® must not be shared, it is for individual use only.
- Always use a new lancet, which must only be placed into the lancing device directly before testing.
- Discard the lancet after testing and do not leave the lancet in the device after use.
- Clean the cap after using it.
- Always dispose of the used lancet in a safe way, e.g. in a puncture resistant container, according to local regulations.
- Keep out of reach of children. This device has many small parts and could be a choking hazard for children if swallowed.

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HOW TO USE MYSTAR SYLKFEEL®

Wash your hands carefully with soap and warm water to increase the circulation of the blood into the fingers. Rinse and thoroughly dry your hands.

- Make sure there is no grease, oil or lotion on the fingertip you are going to use.
- Keep your hand warm or gently massage the site you are lancing to help your blood flow.

Please read the following steps and view the corresponding illustrations to understand how to use the MyStar SylkFeel®. For proper handling of the lancets please check the instruction that came with the lancet.

CLEANING

Never put the body of the MyStar SylkFeel® in water or any other liquid solution. Clean the outside of the MyStar SylkFeel® using a damp cloth with mild cleaning fluid or mild soap. To clean the adjustable cap and the AST clear cap, take the cap off, wash it in warm water, rinse well and dry.

STORAGE

Products must be stored at room temperature protected from sunlight or moisture.

Manufacturer / Fabricante:
HTL-STREFA S.A.
Adamówek 7,
95-035 Ozorków
Poland / Polonia

Distributed by: Sanofi UK
One Onslow Street, Guildford, Surrey, GU1 4YS
All rights reserved. All trademarks and/or copyrights are properties of their respective owners.
U.S. and International patents pending.

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Customer Service:

08000 35 25 25
www.bgstar.co.uk
In case of emergency, contact your healthcare professional or emergency medical response.

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

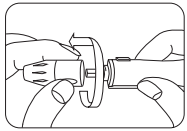
©2013 Sanofi

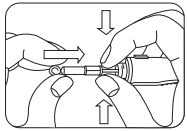
1. Removing the cap: Twist the cap off the lancing device by turning it counter-clockwise while holding the base firmly.

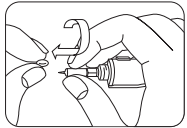
2. Load a new sterile lancet directly into the bottom of lancet carrier.
Note: In case you accidentally cock the device at this point, press the release button before you move on.

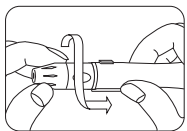
3. While holding the lancet firmly, remove the lancet cap; be careful not to touch the lancet tip.

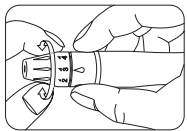
4. Put the cap of the lancing device back on and turn it clockwise until it is fully into place.

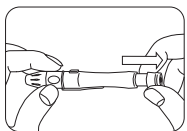
5. The lancing device can be set to the sensitivity of your skin. You can adjust the tip to 5 different lancing depths. Twist the cap end in the appropriate direction until the arrow is pointing to the number for the lancing depth you want.
1–2 for soft or thin skin,
3 for average skin,
4–5 for thick or calloused skin.

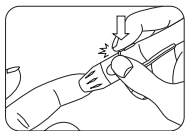
6. Hold the cap firmly in one hand then pull out the sliding barrel with the other hand until it clicks into place. This will cock the lancing device.
Note: Do not lance your finger before the meter and test strips are ready to use.

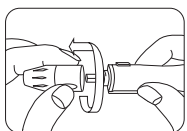7. Hold the MyStar SylkFeel® firmly against the side of the fingertip, with the cap resting on the fingertip.
(Note: the harder you press MyStar SylkFeel® against the finger the deeper the puncture). Press the release button to get a blood sample.

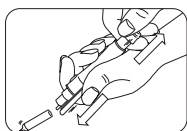8. Removing the cap: Twist the cap off the lancing device by turning it counterclockwise while holding the base firmly.


9. Check the lancet instructions on how to re-cap the lancet. Push the lancet ejector forward with the thumb and simultaneously pull out the sliding barrel to dispose of the used lancet in a safe way, e.g. directly in a puncture resistant container, according to local regulations.
Note: In case the lancet does not fall out, carefully pull the lancet body out of the lancet holder.

AVAILABLE LANCETS


BGStar® and MyStar SylkFeel® lancets are compatible with this device.

USING THE CLEAR CAP FOR ALTERNATIVE SITE TESTING (AST)

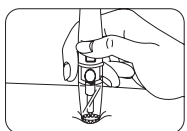(Use also steps 1-4 and 9 from previous instructions)

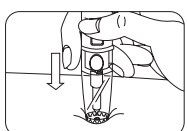1. Place the **MyStar SylkFeel®** on the skin in the chosen area, press and hold the **MyStar SylkFeel®** continuously for a few seconds, then push the release button to take a sample.

2. Watch through the clear cap until a sufficient blood sample is taken.

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524182 - J,138,V81,MYSTAR MG STH,,BDA

Plant: Frankfurt (Germany)
Packaging material code: 524182
Packaging material name: J,138,V81,MYSTAR MG STH,,BDA
Second packaging material code:
VISTAlink folder number: 1192046
VISTAlink PDF version: 1

This document has been digitally signed by the following people within the VISTAlink system, following the sanofi-aventis group guidelines.

Reason	Signed by	Date
Market regulatory validation	Asif Tejpar (UK regulatory team)	27/01/2014 23:29:03
Market regulatory validation	Nieves Santos (Spain regulatory team)	28/01/2014 18:46:29
Plant final technical validation	Christian Albert (Frankfurt Devices Change Service Lead)	30/01/2014 17:44:39
Plant ready to print	Judith Weis (Frankfurt Devices Change Service)	03/06/2014 15:48:40